

Fluorilaq™ Fluorilaq™ Squeeze 5% Sodium fluoride varnish

INDICATIONS FOR USE:

Intended to be used for dental hypersensitivity and as a cavity liner. Fluorilaq is a varnish preparation containing fluoride which may be used to seal dental tubules in cavity preparations or on sensitive root surfaces. Fluorilaq may be used under temporary restoratives and cements where post-operative sensitivity is a concern.

CONTRAINDICATIONS:

Ulcerative gingivitis, stomatitis

WARNINGS:

For topical intraoral use only. Do not ingest. Keep out of reach of children.

HAZARD INFORMATION:

Fluorilaq, as sold, is a flammable material. Once the material is applied and adhered to the teeth Fluorilaq is no longer flammable.

PRECAUTIONS:

- For professional office use only.
- Discontinue patient use of other prescriptive fluoride treatments for 24 hours following Fluorilaq application.
- Avoid use on patients with a known colophony allergy.
- Fluorilaq contains no artificial color, but does contain artificial flavor; confirm that no known patient allergies exist.
- Dyspnea, although extremely rare, has occurred in asthmatic people.
- Nausea may occur when extensive applications have been made to patients with sensitive stomachs.
- If required, varnish can be removed with a thorough brushing.
- Reusing dispensed portion or brushes puts patients at increased risk for injury and infection.

DOSAGE:

For treatment of hypersensitivity: 0.5 grams per quadrant. Two applications within seven days should be adequate for most patients.

For cavity preparation: 0.1 gram per cavity site.

For Fluorilaq Squeeze, each drop contains approximately 20 - 30µL.

INSTRUCTIONS FOR USE:

FOR HYPERSENSITIVE TEETH:

- Remove calculus and excess plaque from teeth.
- Remove excess moisture from area being treated and isolate the area. Formula allows for quick setting in a dry or slightly wet field.
- For Fluorilaq Squeeze, shake bottle before dispensing.
- Dispense required amount of Fluorilaq by either using the "back-of-the-glove" technique or applying to a mixing pad. The "glove" technique frees up one hand to use for cheek retraction, making it faster and easier to apply the varnish.
- Uniformly apply a thin film of Fluorilaq on sensitive areas. Dental floss can be used to reach interproximal areas.
- Remove isolation, allowing saliva or moisture to set Fluorilaq.
- Patient may leave immediately following application of Fluorilaq. Advise patient to not brush and follow a soft food diet for four to six hours.
- After four to six hours Fluorilaq can be removed with a thorough brushing.

AS A CAVITY LINER:

- Clean and dry cavity preparation.
- Remove excess moisture from area being treated and isolate the area. Formula allows for quick setting in a dry or slightly wet field.
- For Fluorilaq Squeeze, shake bottle before dispensing.
- Dispense required amount of Fluorilaq by either using the "back-of-the-glove" technique or applying to a mixing pad. The "glove" technique frees up one hand to use for cheek retraction, making it faster and easier to apply the varnish.
- Uniformly apply Fluorilaq to the prepared area.
- Once dry, apply a second coat of Fluorilaq.
- Once the second coat is dry continue with the procedure.
- Advise patient to not brush and follow a soft food diet for four to six hours.
- After four to six hours Fluorilaq can be removed with a thorough brushing.

All dispensed Fluorilaq should be discarded after use. Brushes should not be reused.

All non-disposable instruments can be cleaned with alcohol to remove Fluorilaq. Normal sterilization procedures should be used.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

Pascal's Fluorilaq is free of gluten, soy, milk, egg, peanut and treenut products.



2929 Northup Way
Bellevue, WA USA 98004
425.827.4694 • 800-426-8051
www.pascaldental.com



SN70105/0418